



GVS Filter Technology UK Ltd.  
Caton Road  
Lancaster  
LA1 3PE  
United Kingdom

Tel: +44 (0) 1524 847600  
Email: gvsuk@gvs.com

EU REP

GVS S.p.A. - www.gvs.com  
Via Roma, 50 - 40069 Zola Predosa, Italy

CE  
2797



**Instructions For Use; Mode d'emploi; Gebrauchsanweisung; Istruzioni per l'uso; Instrucciones de uso; Gebruiksaanwijzing; Instruções de Utilização; Bruksanvisning; Bruksanvisning; Bruksanvisning; Käyttöohjeet; Instrukcja użytkowania; Инструкции за употреба; Upute za upotrebu; Návod k použití; Kasutusjuhend; Οδηγίες χρήσης; Használati útmutató; Treoracha Úsáide; Lietošanas instrukcija; Naudojimo instrukcijos; Istruzioni d'uso; Návod na použití; Navodila za uporabo.**

| REF                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             | A540XXXX<br>Elbow with Sampling Port                                                                                                                                                                                                                                                                                                                                                                                                          | A541XXXX<br>Elbow without Sampling Port                                                                                                                  | A542XXXX<br>Double Swivel Elbow                                                                                                                                       | A543XXXX<br>Double Swivel Elbow                                                                                                                                            |
|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Product Code; Code du produit; Artikelnummer; Codice prodotto; Código de producto; Productcode; Código de produto; Produktkod; Produktkode; Produktcode; Tuotekoodi; Kod produktu; Код на продукта; Šifra proizvoda; Kód výrobku; Tootetood; Код продукта; Termékkód; Cód Táirge; Izstrādājuma kods; Gaminio kodas; Kodići tal-prodott; Cod produs; Kód výrobku; Koda izdelka.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                  |                                                                                                                                                                                                                                                                                                                                                                                                                                               |                                                                                                                                                          |                                                                                                                                                                       |                                                                                                                                                                            |
| Product description; Description du produit; Produktbeschreibung; Descrizione prodotto; Descripción de producto; Productomschrijving; Descrição do produto; Produktbeskrivning; Produktbeskrivelse; Produktbeskrivelse; Tuotteen kuvaus; Opis produktu; Описание на продукта; Opis proizvoda; Popis výrobku; Tootekirjeldus; Περιγραφή προϊόντος; Termékleírás; Tuairisc ar Tháirge; Izstrādājuma apraksts; Gaminio aprašymas; Deskripcija tal-prodott; Descrierea produsului; Popis výrobku; Opis izdelka.<br><b>Elbows; Coudes; Ellbogen; Gomiti; Codos; El-bogen; Cotovelos; Armågar; Albuer; Albu-ene; Kyynärpäät; Kolanka; Колене; Laktovi; Kolena; Torupõlvjed; Γωνίες; Könyök; Ailt Uil-linn; Lıknes; Alkúnés; Gónot; Coate; Kolenka; Komolci.</b>                                                                                                       | Inlet/Outlet Connectors: Male 22mm/Female 15mm ISO 5356 - angled Male 15mm with luer port.<br>Approx. dimensions: 67mm length x 43.5mm width.<br>Weight: 8gm (approx.).                                                                                                                                                                                                                                                                       | Inlet/Outlet Connectors: Male 22mm/Female 15mm ISO 5356 - angled Male 15mm.<br>Approx. dimensions: 53mm length x 43.5mm width.<br>Weight: 8gm (approx.). | Inlet/Outlet Connectors: Male 22mm/Female 15mm ISO 5356 - angled Male 15mm with luer port.<br>Approx. dimensions: 60mm length x 47mm width.<br>Weight: 8gm (approx.). | Inlet/Outlet Connectors: Male 22mm/Female 15mm ISO 5356 - angled Male 15mm with 13mm connector.<br>Approx. dimensions: 50mm length x 47mm width.<br>Weight: 8gm (approx.). |
| Patient population; Population de patients; Patientenpopulation; Popolazione di pazienti; Población de pacientes; Patiëntenpopulatie; População de doentes; Patientpopulation; Patientpopulation; Pasientpopulas-jon; Potilasryhmä; Populacja pacjentów; Популяция пациентов; Populacija pacijenata; Populace pacientů; Patsiendi populatsioon; Πληθυσμός ασθενών; Beteg-populáció; Daonra na n-othar; Pacientu populația; Pacientų populacija; Popolazzjoni ta' pazienti; Populația de pacienți; Pacientov; Populacija bolnikov.                                                                                                                                                                                                                                                                                                                               | No limitation; Aucune limitation; Keine Einschränkung; Nessuna limitazione; Sin limitación; Geen beperking; Sem limitação; Ingen begränsning; Ingen begrænsning; Ingen begrensning; Ei rajoitusta; Bez ograniczeń; Без ограничение; Nema ograničenja; Bez omezení; Piirangid pole; Χωρίς περιορισμό; Nincs korlátozás; Gan teorainn; Bez ierobežojuma; Jokiu apribojimu; Ebda limitazzjoni; Nicio limitare; Žiadne obmedzenie; Brez omejitev. |                                                                                                                                                          |                                                                                                                                                                       |                                                                                                                                                                            |
| <br>Product supplied non sterile; Produit livré non stérile; Das Produkt wird nicht steril geliefert; Prodotto non sterile; Productum suministrado sin esterilizar; Product niet-steriel aangeleverd; Produto fornecido não-estéril; Produkten levereras ej steril; Produkt leveret ikke-sterilt; Produkt leveret ikke-sterilt; Tuote toimitetaan ei-steriilinä; Produkt dostarczany w stanie niejałowym; Продуктът се доставя нестерилин; Proizvod isporučen nesterilan; Výrobek je dodáván v nesterilním stavu; Toode tarnitakse mittesteriilsena; Το προϊόν παρέχεται μη στείρο; A termék nem sterilen szállítva; Soláthraítear an táirge ne-amhsteiriúil; Izstrādājums piegādāts nesterils; Gaminys tiekiamas nesterilus; Prodott fornut mhux steril; Produs furnizat nesteril; Výrobok sa dodáva v nesterilnom stave; Izdelek je dobavljen nesteriliziran. | Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; ДА; Да; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da.                                                                                                                                                                                                                                                                                                                             | Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; ДА; Да; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da.                                        | Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; ДА; Да; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da.                                                     | Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; ДА; Да; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da.                                                          |
| Temperature during storage; Température de stockage; Temperatur bei Lagerung; Temperatura durante l'immagazzinamento; Temperatura durante el almacenamiento; Temperatuur tijdens opslag; Temperatura em armazenamento; Temperatur under förvaring; Temperatur under opbevaring; Temperatur under oppbevaring; Lämpötila varastoinnin aikana; Temperatura podczas składowania; Температура по време на съхранение; Temperatura tijekom skladištenja; Teplota během skladování; Temperatuur ladustamise ajal; Θερμοκρασία κατά την αποθήκευση; Hőmérséklet tárolás közben; Teocht le linn na stórála; Temperatura uzglabšanas laikā; Temperatūra sandėliavimo metu; Temperatura waqt il-hażna; Temperatura în timpul depozitării; Teplota počas składowania; Temperatura med skladiščenjem.                                                                       | 5 to 40°C<br>( 41-104°F)                                                                                                                                                                                                                                                                                                                                                                                                                      | 5 to 40°C<br>( 41-104°F)                                                                                                                                 | 5 to 40°C<br>( 41-104°F)                                                                                                                                              | 5 to 40°C<br>( 41-104°F)                                                                                                                                                   |
| Housing; Boîtier; Gehäuse; Alloggiamento; Carcasa; Behuizing; Estrutura; Kabinet; Hus; Pidike; Obodowa; Kopny; Kucište; Kryt; Úmbris; Περιβλήμα; Burkolat; Tithiocht; Mājokļi; Būstas; Djar; Carcasă; Kryt; Ohșișe.                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             | PP                                                                                                                                                                                                                                                                                                                                                                                                                                            | PP                                                                                                                                                       | PP                                                                                                                                                                    | PP                                                                                                                                                                         |