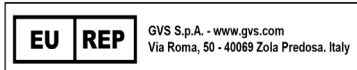
GVS Filter Technology UK Ltd.
Caton Road
Lancaster,
LA1 3PE
United Kingdom

Tel: +44 (0) 1524 847600
Email: gvsuk@gvs.com

GVS S.p.A. - www.gvs.com
Via Roma, 50 - 40089 Zola Predosa, Italy


Instructions For Use; Mode d'emploi; Gebrauchsanweisung; Istruzioni per l'uso; Instrucciones de uso; Gebruiksaanwijzing; Instruções de Utilização; Bruksanvisning; Bruksanvisning; Bruksanvisning; Käyttöohjeet; Instrukcja użytkowania; Інструкція за використання; Upute za upotrebu; Návod k použití; Kasutusjuhend; Οδηγίες χρήσης; Használati útmutató; Treoracha Úsáide; Lietošanas instrukcija; Naudojimo instrukcijos; Istruzioni d'uso; Instrucțiunile de utilizare; Návod na použitie; Navodila za uporabo.

REF	4244/01DEKXXXX Maxi pleat with connectors
Product Code; Code du produit; Artikelnummer; Codice prodotto; Código de producto; Productcode; Código de produto; Produktkod; Produktkode; Produktcode; Tuotekoodi; Kod produktu; Код на продукта; Šifra proizvoda; Kód výrobku; Toote kood; Κωδικός προϊόντος; Termékkód; Cód Táirge; Izstrādājuma kods; Gaminio koda; Kodiči tal-prodott; Cod produs; Kód výrobku; Koda izdelka.	
Customer Product Number; Numéro de produit client; Kundenproduktnummer; Numero prodotto cliente; Número de producto del cliente; Klientproductnummer; Número do produto do cliente; Kundens produkt-nummer; Kundens produktnummer; Kundeproduktnummer; Asiakkaan tuotenumero; Numer produktu klienta; Номер на продукт на клиента; Broj proizvoda kupca; Číslo produktu zákazníka; Kliendi tootenumber; Αριθμός προϊόντος πελάτη; Ügyfél termékszám; Uimhir Táirge Custaiméara; Klienta produkta numurs; Kliento produkto numeris; Numru tal-Prodott tal-Klijent; Numărul produsului client; Číslo produktu zákazníka; Številka izdelka stranke.	
Product description; Description du produit; Produktbeschreibung; Descrizione prodotto; Descripción de producto; Productomschrijving; Descrição do produto; Produktbeskrivning; Produktbeskrivelse; Produktbeskrivelse; Tuotteen kuvaus; Opis produktu; Описание на продукта; Opis proizvoda; Popis výrobku; Tootekir-jeldus; Περιγραφή προϊόντος; Termékleírás; Tuairisc ar Tháirge; Izstrādājuma apraksts; Gaminio aprašymas; Deskrizzjoni tal-prodott; Descrierea produsului; Popis výrobku; Opis izdelka.	Inlet/Outlet connectors: 22mm Male/15mm Female & 22mm Female/15mm Male ISO 5356 and Ø4.3mm luer port. Approx. dimensions: 68.5mm diameter x 81.5mm height. Weight: 40g (approx.). Bidirectional filter. Inlet/Outlet connectors of the catheter mount: - connector 22mm Male / 15mm Female -15mm Male ISO 5356; - connector ISO 5356 22mm Male / 22mm Barbed; - collapsible tube with connectors 15.25mm and 22.25mm (approx.); - elbow: ISO Male 22mm/Female 15mm - angled Male 15mm.
Maxi Vouwfilter met connectors; Maxi plissé avec connecteurs; Maxi-Plisseefilter mit Anschlüssen; Filtro Maxi plissettato con connettori; Maxi Pleat med konnektorer; Maxi Pleat με συνδετήρες; Maxi plissado com conectores; Maxipliegue con conectores; Maxi Pleat, jossa liittimet; Maxi veckat med kopplingar; Maxi skládaný s konektory; Maxi voldik ühendustega; Maxi redős szűrőelem csatlakozókkal; Maxi, plisets ar savienotājēm; “Maxi” plisuotas su jungtimis; Maxi Pleat bil-konnetturi; Maxi Pleat ze złączami; Maxi skládaný s konektormi; Maxi Pleat s priključki; Нагънат филтър Maxi с конектори; Maxi Pléata le nascóirí; Filtru Maxi plisat cu conectori; Maxi nabor s priključcima; Maxi Pleat med koblingsstykker.	
Patient population; Population de patients; Patientenpopulation; Popolazione di pazienti; Población de pacientes; Patiëntenpopulatie; População de doentes; Patientpopulation; Patientpopulation; Pasientpopulasjon; Potilasryhmä; Populacja pacjentów; Популяция пациенти; Populacija pacijenata; Populace pacientů; Patsiendi populatsioon; Πληθυσμός ασθενών; Betegpopuláció; Daonra na n-othar; Pacientu populácia; Pacientų populiacija; Popolazzjoni ta' pazienti; Populația de pacienți; Pacientov; Populacija bolnikov.	Adult; Adulte; Erwachsene; Adulto; Adulto; Volwass-ene; Adulto; Vuxen; Voksen; Voksen; Aikuisen; Dorosly; Възрастен; Odrasla osoba; Dospělý; Tāiskasvanu; Ενήλικος; Felnőtt; Duine Fásta; Pieaugušais; Suaugęs; Adulti; Adult; Dospělý; Odrasla oseba.
	Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; Да; Da; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da
Product supplied non sterile; Produit livré non stérile; Das Produkt wird nicht steril geliefert; Prodotto non sterile; Producto suministrado sin esterilizar; Product niet-steriel aangeleverd; Produto fornecido não-estéril; Produkten levereras ej steril; Produkt leveret ikke-sterilt; Produkt leveret ikke-sterilt; Tuote toimitetaan ei-steriilinä; Produkt dostarczany w stanie niejałowym; Produktът се доставя нестерилен; Proizvod isporučen nesterilan; Výrobek je dodáván v nesterilním stavu; Toode tarnitakse mittesteriilsena; Το προϊόν παρέχεται μη στείρο; A termék nem sterilen szállítva; Soláthraítear an táirge neamhsteriúil; Izstrādājums piegādāts nesterils; Gaminys tiekiamas nesterilus; Prodott forut mhux sterili; Produs furnizat nesteril; Výrobok sa dodáva v nesterilnom stave; Izdelek je dobavljen nesteriliziran.	5 to 40 °C (41-104 °F)
Temperature during storage; Température de stockage; Temperatur bei Lagerung; Temperatura durante l'immagazzinamento; Temperatura durante el almace-namiento; Temperatuur tijdens opslag; Temperatura em armazenamento; Temperatur under förvaring; Temperatur under opbevaring; Temperatur under oppbev-aring; Lämpötila varastoinnin aikana; Temperatura podczas składowania; Температура по време на съхранение; Temperatura tijekom skladištenja; Teplota během skladování; Temperatuur ladustamise ajal; Θερμοκρασία κατά την αποθήκευση; Hőmérséklet tárolás közben; Teocht le linn na stórála; Temperatura uzglabāšanas laikā; Temperatura sandēliavimo metu; Temperatura waqt il-hażna; Temperatura în timpul depozitării; Teplota počas skladovania; Temperatura med skladištenjem.	
Internal Volume; Volume interne; Innenvolumen; Volumen interno; Volume interno; Intern volume; Volume interno; Innervolum; Internt volumen; Indre volum; Sis-ättilavuus; Objętość wewnętrzna; Вътрешен обем; Unutarnji volumen; Vnitřní svazek; Sisemine maht; Εσωτερικός όγκος; Belső kötet; Imleabhar Innheánach; Iekšējais tilpums; Vidinis tūris; Volum Intern; Volum intern; Vnútný objem; Notranja glasnost.	65ml
Resistance; Résistance; Widerstand; Resistenza; Resistencia; Weerstand; Resistans; Modstand; Motstand; Vastus; Ορόρ; Съпротива; Otpornost; Odpor; Vastupanu; Αντίσταση; Ellenállás; Frietoiacht; Pretestība; Pasipriešinimas; Rezištenza; Rezištență; Odpor; Odpornost.	Max.174.9Pa @ 30L/min; Max.392.7Pa @ 60L/min; Max.627Pa @ 90L/min.
Filter Efficiency @ 30l/min; Efficacité du filtre à 30l/min; Filterleistung bei 30l/min; Efficienza del filtro @ 30l/min; Eficiencia del filtro a 30l/min; Filterefficiëntie @ 30l/min; Eficiência do filtro @ 30l/min; Filtreringseffektivitet @ 30l/min; Filtreringseffektivitet @ 30l/min; Filter Effektivitet @ 30l/min; Suodattimen tehokkuus @ 30l/min; Wydajność filtra @ 30l/min; Ефективност на филтъра @ 30l/min; Účinnost filtru @ 30l/min; Účinnost filtru při 30l/min; Filtri efektiivsus @ 30l/min; Αποδοτικότητα φίλτρου @ 30l/min; Szűrőhatékonyaság @ 30l/perc; Éifeachtacht Scagaire @ 30l/nóim; Filtra efektivitate @ 30l/min; Filtro efektyvumas @ 30l/min; Efficjenza tal-filtra @ 30l/min; Efficjenza filtrului @ 30l/min; Účinnost filtra @ 30l/min; Účinnost filtra @ 30l/min.	Min.99.95%
Bacterial Efficiency; Rétenction bactérienne; Bakterienretention; Ritenzione di batteri; Retención de bacterias; Bacterieretentie; Retenção bacteriana; Bakteri-er-entention; Bakterieretention; Bakterieretensjon; Bakteerien suodatus; Zatrzymywanie bakterii; Ефективност на бактериите; Účinnost bakterija; Bakteriální účinnost; Bakterite efektiivsus; Βακτηριακή αποτελεσματικότητα; Bakteriális hatékonyság; Éifeachtúlacht Baictéarach; Baktériju efektivitate; Bakteriju efektyvu-mas; Efficjenza Batterjali; Eficjență bacteriană; Bakteriálna účinnost; Účinnost bakterij.	Min. 99.999%
Viral Efficiency; Rétenction virale; Virenretention; Ritenzione di virus; Retención de virus; Virusretentie; Retenção de virus; Virusretention; Virusreten-tion; Virusretensjon; Virusten suodatus; Zatrzymywanie wirusów; Вирусна ефективност; Virusna učinnost; Virová účinnost; Viiruslik efektiivsus; Ιική αποτελεσματικότητα; Virushatékonyaság; Éifeachtúlacht Víreasach; Vtrusu efektivitate; Virusinis efektyvumas; Efficjenza Virali; Eficjență virală; Vírussová účinnost; Virusna učinnost.	Min.99.999%
Gas leakage; Fuite de gaz; Gasleakage; Perdita de gas; Pérdida de gas; Gaslekkage; Fuga de gases; Gasläckage; Gaslækage; Gasslekkasje; Kaasun vuoto; Uplyw gazu; Изтичане на газ; Propuštanje plina; Unik plynu; Gaasi leke; Διαρροή αερίου; Gázszivárgás; Sceitheadh gáis; Gáznes noplúde; Dujų nuotėkis; Tnixxja tal-gass; Scurgereaz gazului; Unik plynu; Uhajanje plina.	Max.0.0ml/min @ 7kPa
Tidal Volume range; Plage de volume de cycle recommandée; empfohl Hubvolumen-Bereich; Intervallo del volume corrente; Rango de volumen tidal recomen-dado; Aanbevolen slag-volumebereik; Gama do volume corrente recomendado; Rekomenderat andtagsvolymintervall; Anbefalet inspirationsvolumenom-råde; Anbefalt støtolumområde; Suositeltu iskutilavuusalue; Zalecany zakres objętości cyklu; Диапазон на приливния обем; Raspon plimnog volumena; Rozsah dechového objemu; Loodete mahu vahemik; Εύρος παλμορροικού όγκου; Árapály térfogat tartomány; Raon toirt taoide; Plūdmainu apjoma diapazons; Potvyniū tūrio diapazonas; Firxa tal-Volum tal-Marea; Gama volumului mareelor; Rozsah prílivového a odlivového objemu; Obseg dihalne prostornine.	200 - 1500ml
Housing; Boîtier; Gehäuse; Alloggiamento; Carcasa; Behuizing; Estrutura; Cabinet; Hus; Hus; Pidike; Obudowa; Koprny; Kučšte; Kryt; Ümbris; Περιβλήμα; Burkolat; Tithocht; Mäjołki; Büstas; Djar; Carcasă; Kryt; Ohišje.	PP + SBC
Filter Media; Matière du filtre; Filter-Medium; Materiale del filtro; Filtro; Filter medium; Material do filtro; Filtermaterial; Filtermateriale; Filtermedium; Suodattimen väliaine; Medium filtrujące; Филтрираща среда; Filtarski medij; Filtrační prostředí; Filter; Μέσα διήθησης; Szűrőközeg; Meán Scagaire; Filtrirni medij; Filtruoti laikmena; Iffiltra I-Midja; Medii de filtrare; Filtračné médium; Filtrirni medij.	Glass Microfibre Media

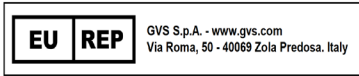GVS Filter Technology UK Ltd.
Caton Road
Lancaster,
LA1 3PE
United Kingdom

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