
Instructions For Use; Mode d'emploi; Gebrauchsanweisung; Istruzioni per l'uso; Instrucciones de uso; Gebruiksaanwijzing; Instruções de Utilização; Bruksanvisning; Bruksanvisning; Bruksanvisning; Käyttöohjeet; Instrukcja użytkowania; Инструкции за употреба; Upute za upotrebu; Návod k použití; Kasutusjuhend; Οδηγίες χρήσης; Használati útmutató; Treoracha Úsáide; Lietošanas instrukcija; Naudojimo instrukcijos; Istruzzjonijiet dwar l-Użu; Instruktioner de utilizare; Návod na použitie; Navodila za uporabo.

REF	1898/01XXXX Scavenger Filter	1898/03XXXX Scavenger Filter	1898/09XXXX Scavenger Filter
Product Code; Code du produit; Artikelnummer; Codice prodotto; Código de producto; Product-code; Código de produto; Produktkod; Produktkode; Produktkood; Tuotekoodi; Kod produktu; Код на продукта; Šifra proizvoda; Kód výrobku; Toote kood; Κυβικός προϊόντος; Termékkód; Cód Táiige; Izstrādājuma kods; Gaminio kodas; Kodići tal-prodott; Cod produs; Kód výrobku; Koda izdelka.			
Customer Product Number; Numéro de produit client; Kundenproduktnummer; Numero prodotto cliente; Número de producto del cliente; Klientproductnummer; Número do produto do cliente; Kundens produktnummer; Kundens produktnummer; Kundeproduktnummer; Asiakaan tuotenumero; Numer produktu klienta; Номер на продукт на клиента; Broj proizvoda kupca; Číslo produktu zákazníka; Kliendi tootenumber; Αριθμός προϊόντος πελάτη; Ügyfél termékzáma; Uimhir Táiige Custaiméara; Klienta produkta numurs; Kliento produkto numeris; Numru tal-Prodott tal-Klijent; Numărul produsului client; Číslo produktu zákazníka; Številka izdelka stranke.			
Product description; Description du produit; Produktbeschreibung; Descrizione prodotto; Descripción de producto; Productomschrijving; Descrição do produto; Produktbeskrivning; Produkt-beskrivelse; Produktbeskrivelse; Tooteleht; Tooteleht; Opis produktu; Описание на продукта; Opis proizvoda; Popis výrobku; Τοτεκίρδus; Περιγραφή προϊόντος; Termékleírás; Tuairisc ar Táiige; Izstrādājuma apraksts; Gaminio aprašymas; Deskripcjoni tal-prodott; Descrerea produsului; Popis výrobku; Opis izdelka.	Inlet/Outlet Connectors of both filters: 22mm Female /15mm Male - 22mm Male /15mm Female ISO 5356. Approx. dimensions: 64.7mm diameter x 94.8mm height (both filters). Weight: 120gm (approx.).	Inlet/Outlet Connectors: 22mm Female /15mm Male - 22mm Male /15mm Female ISO 5356. Approx. dimensions: 64.7mm diameter x 94.8mm height. Weight: 60gm (approx.).	Inlet/Outlet Connectors: 22mm Female /15mm Male - 22mm Male /15mm Female ISO 5356. Approx. dimensions: 64.7mm diameter x 94.8mm height. Weight: 60gm (approx.).
Vangfilter; Filtre d'épuration; Scavenger-Filter; Filtro Scavenger; Skyllefilter; Φίλτρο Scavenger; Filtro sequestrante; Filtro depurador; Scavenger-suodatin; Avluftningsfilter; Filtir pro zachytávání nečistot; Scavenger-filter; Scavenger szűrő; Scavenger filter; "Scavenger" filter; Filtru Scavenger; Filtr scavenger; Filter s neutralizujícíou látkou; Filter za zbiranje; Филтър за улавяне на замърсители; Scagaire Easghlantóra; Filtru Scavenger; Filtar za čiščenje; Scavenger-filter.			
Patient population can range from neonatal to adult. La population de patients peut aller du néonatal à l'adulte. Die Patientenpopulation kann von Neugeborenen bis zu Erwachsenen reichen. La popolazione di pazienti può variare da neonatale ad adulta. La población de pacientes puede variar de neonatal a adulta. Patiëntenpopulatie kan variëren van neonatale tot volwassen. A população de doentes pode variar de neonatal a adulto. Patientpopulationen kan variere från neonatal till vuxen. Patientpopulationen kan variere fra nyfødt til voksen. Potilasryhmä voi vaihdella vastasyntyneestä aikuiseen. Populacja pacjentów może wahać się od noworodków do dorosłych. Популацията на пациентите може да варира от неонатална до възрастна. Populacija pacijenata može biti u rasponu od neonatalne do odrasle osobe. Populace pacientů se může pohybovat od novorozenců až po dospělé. Patsiendi populatsioon võib ulatuda vastsündinutest täiskasvanuteni. Ο πληθυσμός των ασθενών μπορεί να κυμαίνεται από νεογνό έως ενήλικα. A betegpopuláció az újszülöttől a felnőttig terjedhet. Pacientu populácia var būt no jaundzimušā līdz pieaugušajam. Pacientų populacija gali svyruoti nuo naujagimių iki suaugusiųjų. Il-popolazzjoni tal-pazienti tista' tvarja minn trabi tat-twelid sa adulti. Populația pacientului poate varia de la neonatal la adult. Populácia pacientov sa môže pohybovať od novorodeneckých až po dospelých. Populacija bolnikov se lahko giba od neonatalne do odrasle osebe.			
 Product supplied non sterile; Produit livré non stérile; Das Produkt wird nicht steril geliefert; Prodotto non sterile; Producto suministrado sin esterilizar; Product niet-steriel aangeleverd; Produto fornecido não-estéril; Produkten leveras ej steril; Produkt leveret ikke-sterilt; Produkt leveret ikke-sterilt; Tuote toimitetaan ei-steriilinä; Produkt dostarczany w stanie niejałowym; Продуктът се доставя нестерилен; Proizvod isporučen nesterilan; Výrobek je dodáván v nesterilním stavu; Toode tarnitakse mittesteriilsena; Το προϊόν παρέχεται μη στείρο; A termék nem sterilen szállítva; Soláthraítear an táiige neamhsteriúil; Izstrādājums piegādāts nesterils; Gaminys tiekiamas nesterilus; Produkt fornt mhuX steril; Produs furnizat nesteril; Výrobok sa dodáva v nesterilnom stave; Izdelek je dobavljen nesteriliziran.	Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; ДА; Da; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da.	Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; ДА; Da; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da.	Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; ДА; Da; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da.
Temperature during storage; Température de stockage; Temperatur bei Lagerung; Temperatura durante l'immagazzinamento; Temperatura durante el almacenamiento; Temperatur tijdens opslag; Temperatura em armazenamento; Temperatur under förvaring; Temperatur under opbevaring; Temperatur under opbevaring; Lämpötila varastoinnin aikana; Temperatura podczas składowania; Температура по време на съхранение; Temperatura tijekom skladištenja; Toplota během skladování; Temperaturi ladustamise ajal; Θερμοκρασία κατά την αποθήκευση; Hőmérséklet tárolás közben; Teocht le linn na stórála; Temperatura uzglabāšanas laikā; Temperatura sandēliavimo metu; Temperatura waqt il-ħażna; Temperatura în timpul depozitării; Toplota počas skladovania; Temperatura med skladiščenjem.	5 to 40 °C (41-104 °F)	5 to 40 °C (41-104 °F)	5 to 40 °C (41-104 °F)
Resistance; Résistance; Widerstand; Resistenza; Resistencia; Weerstand; Resistência; Resistans; Modstand; Motstand; Vastus; Opór; Съпротива; Odpornost; Odpor; Vastupanu; Αντίσταση; Ellenállás; Friataocht; Pretestība; Pasipriešinimas; Režistenza; Rezištenā; Odpor; Odpornost.	Max. 265Pa @ 30l/min; Max. 732Pa @ 60l/min; Max.1427Pa @ 90l/min.	Max. 208Pa @ 30l/min; Max. 534Pa @ 60l/min; Max.992Pa @ 90l/min.	Max. 160Pa @ 30l/min; Max. 476Pa @ 60l/min; Max.900Pa @ 90l/min.
Filter Efficiency @ 30l/min; Efficacité du filtre à 30l/min; Filterleistung bei 30l/min; Efficienza del filtro @ 30l/min; Eficiencia del filtro a 30l/min; Filtrerefficiëntie @ 30l/min; Eficiência do filtro @ 30l/min; Filtreringseffektivitet @ 30l/min; Filtreringseffektivitet @ 30l/min; Filter Effektivitet @ 30l/min; Suodattimen tehokkuus @ 30l/min; Wydajność filtra @ 30l/min; Ефективност на филтъра @ 30l/min; Üçinkovitost filtra @ 30l/min; Üçinkönni filtru pñ 30l/min; Filtri efektiivsus @ 30l/min; Αποδοτικότητα φίλτρου @ 30l/min; Szűrőhatékonyság @ 30l/perc; Éifeachtacht Scagaire @ 30l/nóim; Filtra efektivitāte @ 30l/min; Filtru efektyvumas @ 30l/min; Efficjenza tal-filtra @ 30l/min; Eficiența filtrului @ 30l/min; Üçinnost filtra @ 30l/min; Üçinkovitost filtra @ 30l/min.	N/A	Min. 94%	N/A
Housing; Boîtier; Gehäuse; Alloggiamento; Carcasa; Behuizing; Estrutura; Cabinet; Hus; Hus; Pidike; Obudowa; Kopnyc; Kucište; Kryt; Umbris; Περιβλήμα; Burkolat; Tithiocht; Mājokļi; Būstas; Djar; Carcasă; Kryt; Ohiše.	SBC	SBC	SBC
Filter Media; Matière du filtre; Filter-Medium; Materiale del filtro; Filtro; Filter medium; Material do filtro; Filtermaterial; Filtermateriale; Filtermedium; Suodattimen väliaine; Medium filtruja; Филтрирающа среда; Filtrarski medij; Filtračni prostredij; Filter; Μέσα διήθησης; Szűrőközeg; Meán Scagaire; Filtrët multividi; Filtruoti laikmena; Iffiltra I-Midja; Medii de filtrare; Filtračné médium; Filtrirni mediji.	Polyester (PET) nonwoven with activated carbon. Activated carbon - granular coconut shell based (black). Alumina based chemisorption media (purple).	Polyester (PET) nonwoven with activated carbon and Electrostatic Blended Synthetic Fiber. Activated carbon - granular coconut shell based (black).	Polyester (PET) nonwoven with activated carbon. Activated carbon - granular coconut shell based (black).

 GVS Filter Technology UK Ltd.
Caton Road
Lancaster
LA1 3PE
United Kingdom
 Tel: +44 (0) 1524 847600
 Email: gvsuk@gvs.com

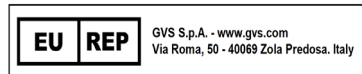Instructions For Use; Mode d'emploi; Gebrauchsanweisung; Istruzioni per l'uso; Instrucciones de uso; Gebruiksaanwijzing; Instruções de Utilização; Bruksanvisning; Bruksanvisning; Bruksanvisning; Käyttöohjeet; Instrukcja użytkowania; Инструкция за употреба; Upute za upotrebu; Návod k použití; Kasutusjuhend; Οδηγίες χρήσης; Használati útmutató; Treoracha Úsáide; Lietošanas instrukcija; Naudojimo instrukcijos; Istruzzjonijiet dwar l-Użu; Instrucțiuni de utilizare; Návod na použitie; Navodila za uporabo.

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