
GVS Filter Technology UK Ltd.
Caton Road
Lancaster,
LA1 3PE
United Kingdom

Tel: +44 (0) 1524 847600
Email: gvsuk@gvs.com

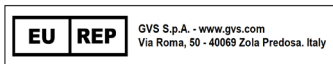Instructions For Use; Mode d'emploi; Gebrauchsanweisung; Istruzioni per l'uso; Instrucciones de uso; Gebruiksaanwijzing; Instruções de Utilização; Bruksanvisning; Bruksanvisning; Bruksanvisning; Käyttöohjeet; Instrukcja użytkowania; Інструкція за використання; Upute za upotrebu; Návod k použití; Kasutusjuhend; Οδηγίες χρήσης; Használati útmutató; Treoracha Úsáide; Lietošanas instrukcija; Naudojimo instrukcijos; Istruzioni d'uso; Instrucciones de uso; Návod na použití; Navodila za uporabo.

REF Product Code; Code du produit; Artikelnummer; Codice prodotto; Código de producto; Productcode; Código de produto; Produktkod; Produktcode; Tuotekoodi; Kod produktu; Код на продукта; Šifra proizvoda; Kód výrobku; Toote kood; Κωδικός προϊόντος; Termékkód; Cód Táirge; Izstrādājuma kods; Gaminio kodas; Kodiči tal-prodott; Cod produs; Kód výrobku; Koda izdelka.	2000/05DLKXXXX Vent Filter Set
Customer Product Number; Numéro de produit client; Kundenproduktnummer; Numero prodotto cliente; Número de producto del cliente; Klientproduktnummer; Número do produto do cliente; Kundens produktnummer; Kundenproduktnummer; Asiakkain tuotenumer; Numer produktu klienta; Номер на продукт на клиента; Broj proizvoda kupca; Číslo produktu zákazníka; Kliendi tootenumber; Αριθμός προϊόντος πελάτη; Ügyfél termék száma; Uimhir Táirge Custaiméara; Klienta produkta numurs; Kliento produkto numeris; Numrul tal-Prodott tal-Klijent; Numărul produsului client; Číslo produktu zákazníka; Številka izdelka stranke.	
Product description; Description du produit; Produktbeschreibung; Descrizione prodotto; Descripción de producto; Productomschrijving; Descrição do produto; Produktbeskrivning; Produktbeskrivelse; Produktbeskrivelse; Tuotteen kuvaus; Opis produktu; Описание на продукта; Opis proizvoda; Popis výrobku; Tootekirjeldus; Περιγραφή προϊόντος; Termék leírás; Tuairisc ar Tháirge; Izstrādājuma apraksts; Gaminio aprašymas; Deskripcija tal-prodott; Descrerea produsului; Popis výrobku; Opis izdelka. Vent Filter Set; Ventilatiefilterset; Jeu filtre d'aération; Beatmungsfiltersatz; Set filtro sfiato; Respirationsfiltersæt; Σετ φίλτρων εξαερισμού; Kit de filtro de ventilação; Juego de filtros de ventilación; Aukon suodattinsarja; Ventilatorfilter set; Sada ventilačních filtrů; Ventilatsioonifiltrikomplekt; Szellőztetőszűrő-készlet; Ventilācijas filtru komplekts; Ventilācijas filtru rinkinys; Sett ta' Filtri tal-Ventilazzjoni; Zestaw filtra wentylacyjnego; Súprava ventilačných filtrov; Komplet filtrov za prezračevanje; Комплект фильтри за механична вентилация; Foireann Scagaire an ghaothaire; Set filtre ventilator; Komplet ventilacijskog filtra; Ventilasjonsfiltersett.	Inlet/Outlet Connectors of the filter 2000/05: OD 8mm (approx.) hose barbed and ID 4.4mm (approx.) - both sides. Approx. dimensions: 54mm diameter x 53.3mm height. Weight: 11g (approx.). Bidirectional filter. Mobile male luer lock connector: OD6.6mm/ID3.8mm, OD ISO 4mm/ID 2.6mm with approx. dimensions: 32mm height. Luer connector: OD 4.4mm (approx.) barbed male luer connector with min. diameter 2.4mm (approx.) thru hole. The length of the luer connector is 22.4mm (approx.). This filter has PVC tubing OD 9mm (approx.) and ID 6mm (approx.) attached to both connectors of the filter, 600mm tubing on one side and 1500mm tubing on the other side. Luer connector is attached to the end of the 600mm tubing and mobile male luer lock connector is attached to the end of the 1500mm tubing.
Patient population; Population de patients; Patientenpopulation; Popolazione di pazienti; Población de pacientes; Patientenpopulatie; População de doentes; Patientpopulation; Patientpopulation; Pasientpopulasjon; Potilasyhmä; Populacja pacjentów; Популяция пациенты; Populacija pacienata; Populace pacientů; Patsiendi populatsioon; Πληθυσμός ασθενών; Betegpopuláció; Daonra na n-oth-ar; Pacientu populația; Pacientų populacija; Popolazzjoni ta' pazienti; Populāja de pacientī; Pacientov; Populacija bolnikov.	No limitation; Aucune limitation; Keine Einschränkung; Nessuna limitazione; Sin limitación; Geen beperking; Sem limitação; Ingen begränsning; Ingen begrænsning; Ingen begrensning; Ei rajoitusta; Bez ograniczeń; Без ограничение; Nema ograničenja; Bez omezení; Piirangid pole; Χωρίς περιορισμό; Nincs korlátozás; Gan teorainn; Bez ierobežojuma; Jokiä pribojmu; Edda limitazzjoni; Nicio limitare; Žiadne obmedzenie; Brez omejitev.
 Product supplied non sterile; Produit livré non stérile; Das Produkt wird nicht steril geliefert; Prodotto non sterile; Producto suministrado sin esterilizar; Product niet-steriel aangeleverd; Produto fornecido não-estéril; Produkten levereras ej steril; Produkt leveret ikke-sterilt; Produkt leveret ikke-sterilt; Tuote toimitetaan ei-steriilinä; Produkt dostarczany w stanie niejałowym; Продуктът се доставя нестерилинен; Προϊζοδισπορευμένο νεστεριλιν; Výrobek je dodávan v nesterilním stavu; Toode tarnitakse mittesteriilsena; Το προϊόν παρέχεται μη στείρο; A termék nem sterilizált; Soláthraítear an táirge neamhsteiriúil; Izstrādājums piegādāts nesterils; Gaminys tiekiamas nesterilus; Prodott fornut mhux sterili; Produs furnizat nesteril; Výrobok sa dodáva v nesterilnom stave; Izdelek je dobavljen nesteriliziran.	Yes; Oui; Ja; Si; Si; Ja; Sim; Ja; Ja; Ja; Joo; Tak; Да; Da; Ano; Jah; Nai; Igen; Tá; Já; Taip; Iva; Da; Áno; Da.
Temperature during storage; Température de stockage; Temperatur bei Lagerung; Temperatura durante l'immagazzinamento; Temperatura durante el almacenamiento; Temperatur tijdens opslag; Temperatura em armazenamento; Temperatur under förvaring; Temperatur under opbevaring; Temperatur under opbevaring; Lämpötila varastoinnin aikana; Temperatura podczas składowania; Температура по време на съхранение; Temperatura tijekom skladištenja; Teplota během skladování; Temperatur ladustamise ajal; Θερμοκρασία κατά την αποθήκευση; Hőmérséklet tárolás közben; Teocht le linn na stórála; Temperatura uzglabšanas laikā; Temperatūra sandėliavimo metu; Temperatura waqt il-hażna; Temperatura în timpul depozitării; Teplota počas skladovania; Temperatura med skladiščenjem.	5 to 40 °C (41-104 °F)
Air Flow; Flux d'air; Luftstrom; Flusso d'aria; Flujo de aire; Luchstroom; Fluxo de ar; Luftflöde; Luftstrøm; Luftstrøm; Ilmavirta; Przepływ powietrza; Въздушен поток; Protok zraka; Proud vzduchu; Öhuuool; Ποή αέρα; Légáramlás; Aershreabhadh; Gaisa plūsma; Oro srautas; Fluss tal-arja; Debit de aer; Prietok vzduchu; Pretok zraka.	Min.85l/min @ 5 PSI
Filter Efficiency @ 15l/min; Efficacité du filtre à 15l/min; Filterleistung bei 15l/min; Efficienza del filtro @ 15l/min; Eficiencia del filtro a 15l/min; Filterefficiëntie @ 15l/min; Eficiência do filtro @ 15l/min; Filtreringseffektivitet @ 15l/min; Filtreringseffektivitet @ 15l/min; Filter Effektivitet @ 15l/min; Suodattimen tehokkuus @ 15l/min; Wydajność filtra @ 15l/min; Ефективност на филтъра @ 15l/min; Učinkovitost filtra @ 15l/min; Účinnost filtru při 15l/min; Filtri efektiivsus @ 15l/min; Αποδοτικότητα φίλτρου @ 15l/min; Szűrőhatékonyaság @ 15l/perc; Éifeachtacht Scagaire @ 15l/nóim; Filtra efektiivtate @ 15l/min; Filtro efektyvumas @ 15l/min; Efficjenza tal-Iffiltra @ 15l/min; Eficjenča filtrului @ 15l/min; Účinnost filtra @ 15l/min; Učinkovitost filtra @ 15l/min.	Min.99.99%
Bacterial Efficiency; Rétention bactérienne; Bakterienretention; Ritenzione di batteri; Retención de bacterias; Bacterieretentie; Retenção bacteriana; Bakterieretention; Bakterieretention; Bakterieretention; Bakteerien suodatus; Zatrzymywanie bakterii; Εφektivnost na bakterijite; Učinkovitost bakterija; Bakteriální účinnost; Bakterite efektiivsus; Βακτηριακή αποτελεσματικότητα; Bakteriális hatékonyság; Éifeachtúlacht Baictéarach; Baktériju efektiivtate; Baktériju efektyvumas; Efficjenza Batterjali; Efficjenča bacteriañ; Bakteriálna účinnost; Učinkovitost bakterij.	Min.99.999%
Viral Efficiency; Rétention virale; Virenretention; Ritenzione di virus; Retención de virus; Virusretentie; Retenção de vírus; Virusretention; Virusretensjon; Virusten suodatus; Zatrzymywanie wirusów; Вирусна ефективност; Virusna učinkovitost; Virová účinnost; Viiruslik efektiivsus; Ιική αποτελεσματικότητα; Virushatékonyaság; Éifeachtúlacht Víreasach; Virusu efektiivtate; Virusinis efektyvumas; Efficjenza Virali; Eficjenča viralá; Virusová účinnost; Virusna učinkovitost.	Min.99.999%
Burst pressure; Pression d'éclatement; Berstdruck; Pressione di scoppio; Presión de ráfaga; Barstdruk; Pressão de estouro; Pressão de estouro; Brud tryk; Burst tryk; Murtumispaine; Císnienie pęknięcia; Спукано наляганне; Pritsak pucanja; Tlak při roztržení; Lõhkemissurve; Πίεση ρήξης. Robbanási nyomás; Brú pléascátha; Párraušanas spiediens; Trūkio slėgis; Pressjoni tat-tifqigh; Presiune de spargere; Tlak prasknutia; Tlak v rupturi.	Min. 64psi
Housing; Boîtier; Gehäuse; Alloggiamento; Carcasa; Behuizing; Estrutura; Kabinet; Hus; Hus; Pidike; Obudowa; Kopnyc; Kucište; Kryt; Úmbris; Περιβλήμα; Burkolat; Tithocht; Májokji; Büstas; Djar; Carcasă; Kryt; Ohišje.	PP+ABS+PVC tubing
Filter Media; Matière du filtre; Filter-Medium; Materiale del filtro; Filtro; Filter medium; Material do filtro; Filtermateriale; Filtermedium; Suodattimen väliaine; Medium filtrujące; Filtrirača sreda; Filtrarski medij; Filtrační prostředi; Filter; Μέσα διήθησης; Szűrőközeg; Meán Scagaire; Filtrēt multividi; Filtruoti laikmena; Iffiltra l-Midja; Medii de filtrare; Filtračné médium; Filtrirni mediji.	Hydrophobic Glass Microfibre Media

 GVS Filter Technology UK Ltd.
Caton Road
Lancaster,
LA1 3PE
United Kingdom
 Tel: +44 (0) 1524 847600
 Email: gvsuk@gvs.com

Instructions For Use; Mode d'emploi; Gebrauchsanweisung; Istruzioni per l'uso; Instrucciones de uso; Gebruiksaanwijzing; Instruções de Utilização; Bruksanvisning; Bruksanvisning; Bruksanvisning; Käyttöohjeet; Instrukcja użytkowania; Инструкция за употреба; Upute za upotrebu; Návod k použití; Kasutusjuhend; Οδηγίες χρήσης; Használati útmutató; Treoracha Úsáide; Lietošanas instrukcija; Naudojimo instrukcijos; Istruzzjonijiet dwar l-Użu; Instrucțiuni de utilizare; Návod na použitie; Navodila za uporabo.

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